




Maxi-Flex Flexible Ureteroscope "Flexi-Scope"

Specifications

- * Semi-Disposable Flexible Ureteroscope
- * Model as per the requirements, which can be used in Adult & Pediatric both:
 - Model ENDO-45 (Working length 45cm) : For Pediatric Patient
 - Model ENDO-55(Working length 55cm) & ENDO-65(Working length 65cm) : For Adult Patient

Note: Eyepiece will be common for both Scopes

- * Working Channel : 1.16mm OR 3.5 Fr.
- * Distal Tip Diameter : 2.1mm OR 6.3 Fr.
- * Irrigation Channel : 3.5 Fr.
- * Field of View : 70 Degree on Air/ 58 Degree on Water
- * Direction of View : 0 Degree
- * Depth of View: 2- 12 mm
- * Active Deflection: 180 Degrees Bi-Directionally
- * Recommended Guide wire : <0.035"
- * Fully flexible tip Length: 8 cm
- * Reusable Eyepiece
- * Standard Light Source connector
- * Dual defection
- * Radio opaque distal tip

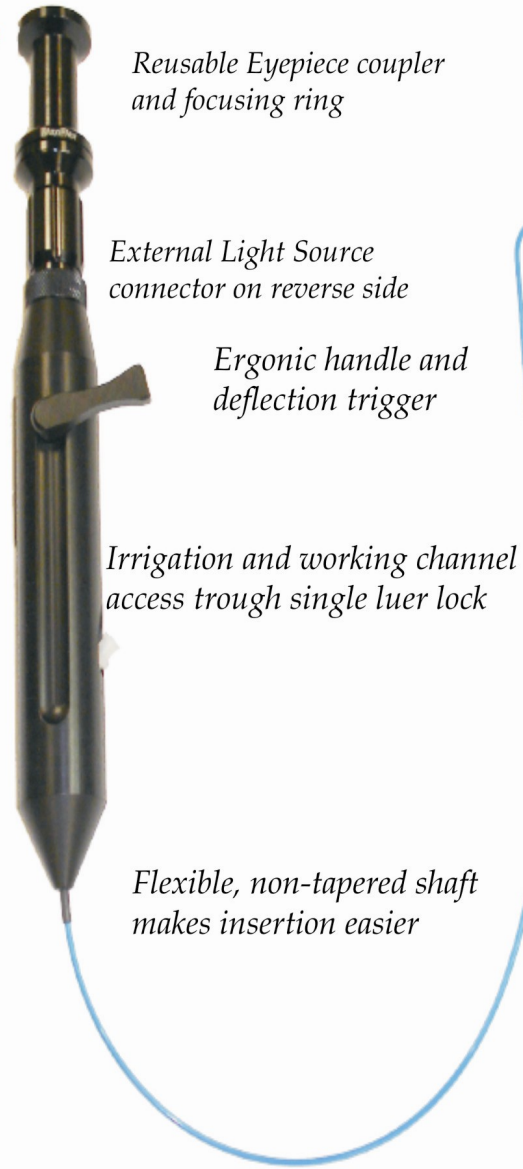

Reusable Eyepiece coupler and focusing ring

External Light Source connector on reverse side

Ergonic handle and deflection trigger

Fully flexible tip with Pt/Ir. Band for imaging with fluroscopy

Irrigation and working channel access trough single luer lock

Flexible, non-tapered shaft makes insertion easier

Manufactured By:



MaxiFlex LLC.
 LSU Louisiana Emerging Technology Center
 Building 340, East Parker St. Ste. 240
 BatonRouge, LA 70803
 phone: 866-629-4359
 fax: 225-615-8939
 www.maxiflexllc.com



MaxiFlex
Advanced Surgical Devices
for Urology

Semi-Flex Scope

The Flexi-scope, although is very capable, is not intended to be a complete replacement of your existing reusable flexible ureteroscope. Instead, you should formulate a strategy to incorporate the semi-disposable scope to perform certain procedures to minimize the risk of damages of your existing expensive reusable scope & also minimize the total cost by reducing the frequency of costly repair. To facilitate this, a patient selection procedure should be developed to assist the physician in deciding when to use the Flexi-Scope. The selection procedure should be specific to the facility that you are in, the experience the doctor has with the Flexi-Scope, The condition of your reusable scopes and how aggressively you wish to cut costs. We recommend a conservative approach for doctors who are new to the Flexi-Scope primarily targeting patients who have:

- 1.) Solitary renal stone less than or equal to 7mm in the upper or mid pole
- 2.) Solitary uretral stone less than or equal to 7mm in the upper or mid ureter
- 3) Less than 5 small renal stones (Less Than or equal to 5mm) in the upper or mid pole
- 4.) Stone in the lower calyx should only be attempted by physicians who have prior experience with the Flexi-Scope in the middle & upper calyces
- 5) anatomic variations that may prevent successful surgery should be excluded such as significant hydronephrosis, calyceal diverticulum, etc.

Starting with this strategy, the facility & doctor can develop their own patient selection criteria as the experience and confidence of the physician with the Flexi-Scope grows. The use of the Flexi-Scope may be advantages during rapid Patient turn around, or when your reusable scope is out for repair.

In some private pay cases, devices intended for 4-5 time use may be reimbursed. Contact a Maxi-Flex Representative M/s Unique Medical Devices for further details.

Advantages:

- Eliminates Capital purchases and costly repairs associated with reusable scopes
- Lessens damages for your more expensive scopes needed for difficult cases
- Rigid enough to pass through the ureteral orifice
- 6.3 Fr slim design may obviate dilation, thereby shortening or time and time under anesthesia
- Due to reprocessing and normal wear, surgeons may not be aware of damage to a reusable scope until surgery
- With the Flexi-Scope you will have a brand new scope everytime after few uses
- Back to back surgeries will never be problem since you will always have a new scope waiting for the next case
- Working channel accepts < 0.035" Guide wires & other devices < 0.035" or smaller
- Reduces rescopes infections because it can be used upto 4-5 cases of little more
- Reduces cost of procedure to Hospital
- Billable to patient non-medicare/ ad
- Small enough to pass through and access sheath without removing the guide wire

Contact Authorized Distributor



Unique Medical Devices (Customer Care: 011- 27296595)

HO: Off No. 11, 2nd Floor, New Gurgaon Plaza market, Opposite-sector 14

ICICI Bank, Old Delhi Road, Gurgaon - 122 001 (Haryana). INDIA

Ph.: 0124-4080411/ Fax. : 0124-4384755. E-mail: info.umd@gmail.com

website: www.umd.ind.in